



Length measurement

· Measurement of the difference between foot and shoe length (mm)

Width measurement

· Measurement of the difference between foot and shoe width (mm)

The following four symbols indicate how the shoe fits:



12 - 16 mm space

Shoe fits perfectly

Ideal for new shoes



6 - 12 mm space

Shoe is slightly too small

Consider changing



More than 16 mm space

Shoe is too big

Oversized shoes, do not wear



Below 6 mm space

Shoe is too small

Change shoes as soon as possible

Quick guide: Orthopaedic children's foot and shoe measuring device



CLEVERMESS
...the clever way to a fitting shoe!

Preparing



1. Switch on the device. A beep will sound.



2. For shoe sizes over 31, first attach the extension. Place toe and heel stop on the device.



3. Pull the device apart to the approximate foot length by simultaneously pressing the side buttons.



4. Place the child's foot straight & firmly on the measuring device. Adjust the length setting if necessary, so that the heel and toes stop exactly against the device.

Measuring foot length

5. Insert the width clamp above the foot into the side recesses and connect them at the top.



6. Press the width clamp together to the sides of the foot whilst pressing in the front measuring tongue until two beeps sound.



Measuring foot width

7. The display will flash (it won't show the measurements!). Remove attachments and insert the measuring device into the desired shoe. When measurement is complete, the unit will beep 3 times and display result for 40 sec.



8. After the finishing tone (five short beeps), a new shoe can now be measured. Remember to switch off the device after use.



Comparing shoe